Dieses Produkt ist für die mehrfache Verwendung durch eine Person bestimmt. / This product is intended for multiple use by one person. / Ce produit est destiné à un usage multiple par une seule personne. / Questo prodotto è destinato all'uso multiplo da parte di una sola persona.

Stand 01.10.2023

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## GB Classification

**MD** In accordance with the classification regulations in accordance with Annex VIII, these are Class 1 medical devices.

### Explanation

We hereby declare in our sole responsibility that this class 1 medical device complies with the basic requirements of regulation EU 2017/745 Annex 1.

### Warranty

We warrant this product against defects in materials and workmanship, for a period of 24 months.

### Material:

Upper material: 100% polyamide  
Lining: 100% terry  
Lamination: duplex membrane  
Inner Equipment: antibacterial / silver ions  
Counter outside: synthetic leather  
Velcro tape: textile  
Insole: Soft foot impression foam  
upper side 100% terry  
PU-sole / hand-stitched

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## GB Instructions

With this medical aid you have purchased a post-operative shoe of a high quality.

**Scope of delivery:** 1 pair of post-operative shoes with instructions.

**Purpose:** Post-operative shoe for long-term use to accommodate the foot provided with a wound or padding dressing.

**Indication:** Postoperative or other wounds, also provided with expanded, where required cushioned bandages, whose healing process proceeding over a longer period of time and allows at least a partial weight bearing of the foot when walking, outside as well. Additional bedding is not required.

**Contraindication:** Sensation disorders and circulatory disturbances in the feet.

**Product characteristics:** Post-operative shoe for long-term use, which can be opened sufficiently wide and has enough volume to hold the bandaged foot and possible bedding. The material allows a soft and flexible adjustment for nearly every foot shape. Individual adjustable fasteners. Non-slip outsole, removable insole, suited for indoor and outdoor areas.

**Important advice:** Avoid a disruption of the blood circulation through closing the shoe too tight. The shoe is not suited for reuse and only to be used by one person. The shoe must not be worn if one has open wounds. The shoe must only be used when intact. Be careful to close the shoe tight, but not too tight. Closing the shoe too loose or too tight might reduce the therapeutic effect or cause injuries. If you face any problems with your feet please contact a physician. Be careful in the event of ice and snow. Not waterproof.

**Finish of the upper:** Tailoring, sewing and new overcasting is possible.

**Finish of the bottom area:** Possible to adapt the sole through grinding and gluing adhesive bonding. Adjustments may only be made by trained orthopedic shoe technicians.

**Application:** Possibility of wide opening and individually adjustable fastener. Refrain from wearing the shoe without the provided or individually made insole. Make sure to scan the inside of the shoes before wearing them and remove any foreign objects.

**Care instructions:** Shoes are washable up to 40°C. Please avoid abrasive materials such as brushes, steel wool or sandpaper. Do not use any aggressive products such as alcohol, bleach, ammonia or solvents. After cleaning let the shoe air dry. Do not let it dry in or on direct sources of heat. Disinfection with common household products by spray disinfection.

**Product disposal:** The shoes must be disposed of in compliance with local provisions. Please do not cast into nature after use.

**Questions and problems:** For questions about the product or problems with your post-operative shoe contact your specialist dealer or orthopedic shoe technician. All serious incidents that have occurred in connection with the product must be reported to the manufacturer (see reverse side) and the authority responsible for the user.

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