Dieses Produkt ist für die mehrfache Verwendung durch eine Person bestimmt. / This product is intended for multiple use by one person. / Ce produit est destiné à un usage multiple par une seule personne. / Questo prodotto è destinato all'uso multiplo da parte di una sola persona.

Stand 01.06.2024

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## GB Classification

**MD** In accordance with the classification regulations in accordance with Annex VIII, these are Class 1 medical devices.

### Explanation

We hereby declare in our sole responsibility that this class 1 medical device complies with the basic requirements of regulation EU 2017/745 Annex 1.

### Warranty

We warrant this product against defects in materials and workmanship, for a period of 24 months.

### Material:

Upper material: leather / textile / stretch  
Upper material Model 600 noir: genuine leather  
Upper material Model 640: genuine leather  
Lining: microfibre "OnSteam"  
Lining Model 600: textile  
Lining Model 640: virgin wool  
Lamination: moltopren foam  
Stretch material: Lycra (80% PA / 20% EA)  
Insole: microfibre / EVA-foam  
Sole: PU-sole / hand-stitched

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## GB Instructions

With this medical aid you have purchased a post-operative shoe of a high quality.

**Scope of delivery:** 1 pair of post-operative shoes with instructions.

**Purpose:** Post-operative shoe for long-term use to accommodate the foot pro- vided with a wound or padding dressing.

**Indication:** Postoperative or other wounds, also provided with expanded, where required cushioned bandages, whose healing process proceeding over a longer pe- riod of time and allows at least a partial weight bearing of the foot when walk- ing, outside as well. Additional bedding is not required.

**Contraindication:** Severe diabetes, correcting osteotomies not to be strained, open wounds, hammer toe D II and D III.

**Product characteristics:** Post-operative shoe for long-term use, which can be opened sufficiently wide and has enough volume to hold the bandaged foot and possible bedding. The stretch-material allows a soft and flexible adjustment for nearly every foot shape. The expandable material moves along with every movement and prevents painful pressure marks. Non-slip outsole, removable insole, suited for indoor and outdoor areas.

**Important advice:** Avoid a disruption of the blood circulation through closing the shoe too tight. The shoe is not suited for reuse and only to be used by one person. The shoe must not be worn if one has open wounds. The shoe must only be used when intact. Be careful to close the shoe tight, but not too tight. Clos- ing the shoe too loose or too tight might reduce the therapeutic effect or cause injuries. If you face any problems with your feet please contact a physician. Be careful in the event of ice and snow. Not waterproof.

**Special property:** by additionally available footbeds, the bale dimensions and the toe heights can be increased per foot side, either -left- or -right-. After re- covery of the foot, everything can gradually again be made backward.

**Finish of the upper:** Tailoring, sewing and new overcasting is possible.

**Finish of the bottom area:** Possible to adapt the sole through grinding and gluing adhesive bonding. Adjustments may only be made by trained orthopedic shoe technicians.

**Application:** Dressing shoe made of leather and flexible material. Wide opening pos- sibilities and individually adjustable closures. Do not wear the shoe without the included or custom made insole. Flatten shoes inside before removing them and remove any foreign objects.

**Care instructions:** Clean the shoe with standard shoe care products for leather and textiles. Please avoid abrasive materials such as steel wool, sandpaper. Do not use aggressive products such as alcohol, ammonia and solvents. Disinfection with common household products by spray disinfection.

**Product disposal:** The shoes must be disposed of in compliance with local provisions. Please do not cast into nature after use.

**Questions and problems:** For questions about the product or problems with your post-operative shoe contact your specialist dealer or orthopedic shoe technician. All serious incidents that have occurred in connection with the product must be reported to the manufacturer (see reverse side) and the authority responsible for the user.

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