Dieses Produkt ist für die mehrfache Verwendung durch eine Person bestimmt. This product is intended for multiple use by one person. / Ce produit est destiné à un usage multiple par une seule personne. / Questo prodotto è destinato all'uso multiplo da parte di una sola persona.

Stand 01.11.2023

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GB Classification

In accordance with the classification regulations in accordance with Annex VIII, these are Class 1 medical devices.

Explanation

We hereby declare in our sole responsibility that this class 1 medical device complies with the basic requirements of regulation EU 2017/745 Annex 1.

Warranty

We warrant this product against defects in materials and workmanship, for a period of 24 months.

Material:

Upper material: 100% polyester
Lining: 100% polyester
Lamination: moltopren foam
Counter outside: synthetic leather
Velcro tape: synthetic leather
Insole: EVA-foam
Sole: PU anti-slip sole

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GB Instructions

With this aid you have purchased a high-quality therapeutic shoe, which helps the relief of the forefoot area and early mobilisation. The Product keeps all of his functions if used properly and as long as the shoe stays intact and is not showing any signs of excessive wear.

Scope of delivery: 1 therapy shoe with instructions for use.

Purpose: Therapy shoe for holding the foot with a wound or padding bandage.

Indication: Post-operative with forefoot operations, wounds and injuries in the forefoot area.

Contraindication: None if used properly.

Product characteristics: The forefoot relief shoe 860 has the following features:

- wide opening option.
- individually adjustable closure.
- the shoe has a continuous footplate that extends up to reaches to the toes and protects them from impacts.
- Full-length outsole with anti-slip protection.
- Secure fixation of the shoe through the shaft and Locking mechanism.

Important advice: Avoid a disruption of the blood circulation through closing the shoe too tight. The shoe is not suited for reuse and only to be used by one person. The shoe must not be worn if one has open wounds. The shoe must only be used when intact. Be careful to close the shoe tight, but not too tight. Closing the shoe too loose or too tight might reduce the therapeutic effect or cause injuries. If you face any problems with your feet please contact a physician. Be careful in the event of ice and snow. Not waterproof.

Finish of the upper: not intended.

Finish of the bottom area: not intended.

Application: Wear the shoe only if instructed by a physician. Protect your injury or post-operative wounds with a bandage. If necessary, use walking aids. Make sure to always wear a non-slip shoe on your healthy foot. Do not drive while wearing the Forefoot-relief-shoe. The shoe can be worn indoors and outdoors.

Care instructions: Handwash the shoe using lukewarm water and a mild detergent. Please avoid abrasive materials such as brushes, steel wool or sand-paper. Do not use any aggressive products such as alcohol, bleach, ammonia or solvents. After cleaning let the shoe air dry. Do not let it dry in or on direct sources of heat. Avoid direct sunlight. Disinfection with common household products by spray disinfection.

Product disposal: The shoes must be disposed of in compliance with local provisions. Please do not cast into nature after use.

Questions and problems: For questions about the product or problems with your post-operative shoe contact your specialist dealer or orthopedic shoe technician. All serious incidents that have occurred in connection with the product must be reported to the manufacturer (see reverse side) and the authority responsible for the user.

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